



## An Important Reminder: ImpaCt Xp Volume Control Function

When the instrument is first turned on, the gain is set to the programmed *Target Gain* regardless of the amount of VC rotation.

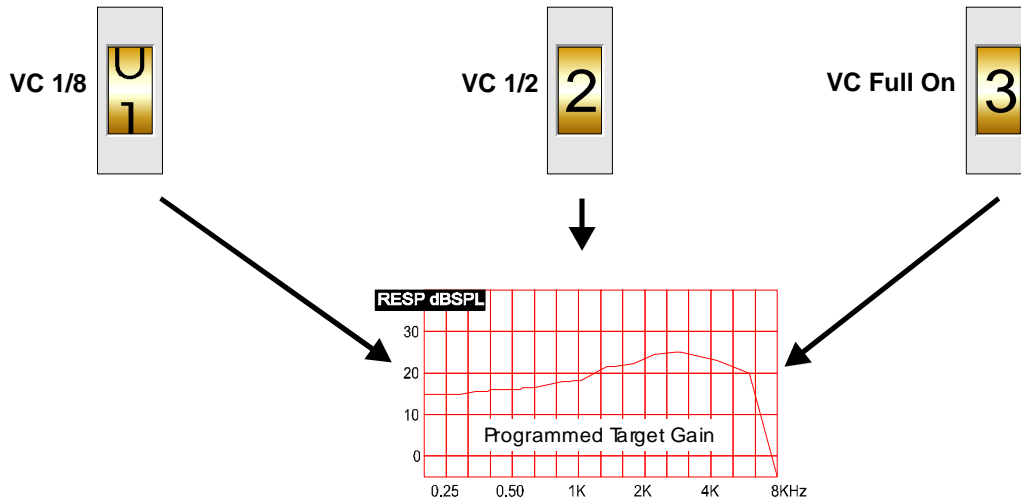

All three VC positions equal *Target Gain* at start-up.

After one second the volume control becomes active and any subsequent volume control rotation will adjust the gain to match the new volume control setting.

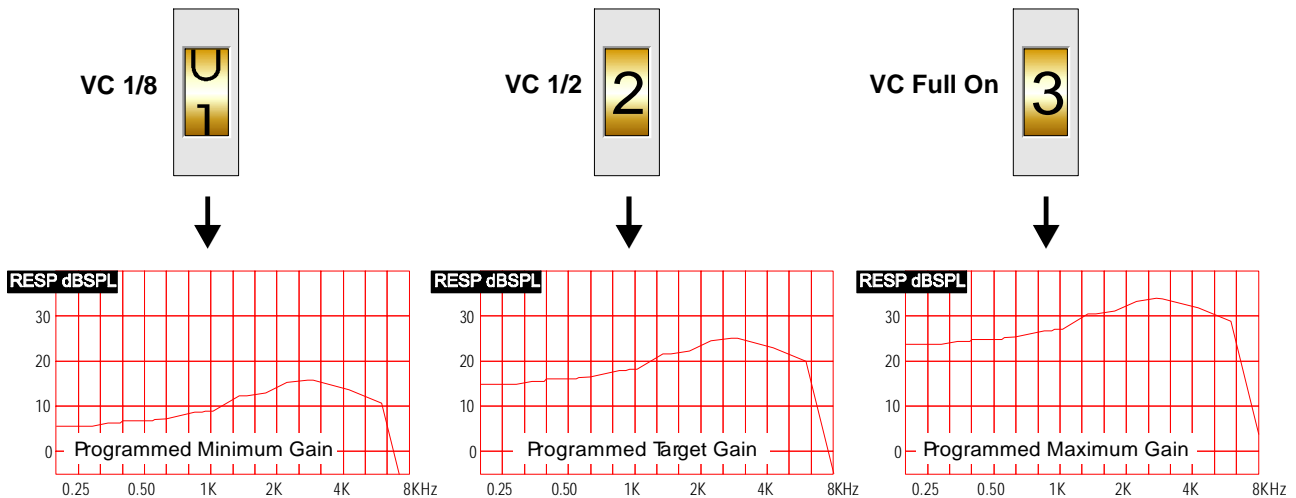

Note: Examples are based on 18 dB Volume Control Range and a randomly selected audiogram.

### Fitting Tip:

Urge your client to develop the habit of turning the Volume Control to 1/2 at start-up. It is a close approximation of their target gain setting and it allows them the familiarity of turning up for more volume and down for less.